

Neonatal Intubation Trainer



ENGLISH

Directions for Use

Table of Contents

Introduction2

Laerdal Recommends2

Items Included2

Skills Taught.....2

Instructions for Use.....2

Care and Maintenance.....2

Replacement Parts2

Introduction

The Laerdal Neonatal Intubation Trainer (250-00101) is a lifelike manikin head which realistically simulates that of a newborn baby. It is specifically designed for training professionals in the practice of oral and nasal intubation.

The manufacturing quality and ease of replacing individual parts of this trainer should provide many years of training when reasonable care and maintenance is practiced.

Laerdal Recommends

Uncuffed endotracheal tube (internal diameter 2.5mm)

Manikin Lubricant or liquid soap

Laryngoscope handle

Laryngoscope blade #0 or #1



Caution: Latex

This product contains **Natural Rubber** latex which may cause allergic reactions when in contact with humans.

Items Included

- (1) Laerdal Neonatal Intubation Trainer
- (1) Manikin Lubricant
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Skills Taught

- Oral and nasal intubation
- Bag-Valve Mask ventilation
- Correct Tube placement

Instructions for Use

For intubation, the following equipment is needed:

- Uncuffed endotracheal tube (internal diameter 2.5mm)
- Lubricant (or liquid soap)
- Laryngoscope handle
- Laryngoscope blade #0 or #1

1. Spray all tubes with Manikin Lubricant prior to airway placement.
2. Spray the inside of each nostril with Manikin Lubricant prior to nasal intubation.

Care and Maintenance

1. Clean with mild soap and water; do not submerse manikin or parts in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin and module should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Store properly between teaching sessions.

Replacement Parts

Please visit www.laerdal.com.

Notes: